



Instructions for use HeltiQ Skintags

- Before use carefully read the instructions for use - For external use only
- Indicated for use on adults and children over 4 years- The method for treating skintags is cryotherapy that involves freezing the skin tag -



Instructions for use HeltiQ Skintags


- Before use carefully read the instructions for use - For external use only
- Indicated for use on adults and children over 4 years- The method for treating skintags is cryotherapy that involves freezing the skin tag -

REF: 146295
741157 - 03.01.06517

CONTENT INSTRUCTION FOR USE

1. Content HeltiQ Skintags
2. Diagnose
3. Instructions for use
4. After the treatment
5. Side effects
6. Contraindications
7. Warnings and precautions

1. CONTENT HELTIQ SKINTAGS

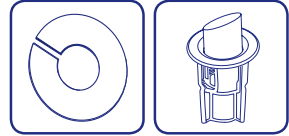

- 1 Aerosol dispenser with cap, containing 38 ml dimethyl ether
- 1 Instruction for use
- 8 Protective plasters
- 8 Single use disposable foam tips in holder

2. DIAGNOSE SKIN TAGS



Skin tags are also known as acrochordons and occurs frequently in both men and women. Skin tags are benign and non-contagious. They can occur spontaneously, usually in the armpits, on the neck and throat area or under the breasts. HeltiQ Skintags can only be used on these parts of the body.

A skin tag has the following characteristics:

- Skin colored
- Soft and painless
- Moves easily
- Protrudes
- Has a small stem



CAUTION!

In most cases, a single treatment is sufficient to remove the skin tag. In case of multiple skin tags, located close to each other, treat each skin tag separately with an interval of one day in-between treatments. When the skin tag is still present after two weeks, repeat the treatment. Do not treat a skin tag more than four times within a treatment interval of two weeks. If this treatment regime fail to have the desired effect, it is best to seek medical advice.

3. INSTRUCTIONS FOR USE

Do not remove the plastic cap from the pressurized aerosol dispenser.



Place a plaster around the skin tag to be treated.



Place a foam tip holder in the opening of the cap. Press the holder gently into the opening. Hold the dispenser upside down with the holder and foam tip pointing downwards towards the floor. Do not place the foam tip on the skin tag.



Activate the dispenser and make the foam tip active. Hold the product away from your body and do not point in the direction of persons in the room. Press four times firmly with the thumb on the bottom of the dispenser. The content will flow into the foam tip. Hold the dispenser upside down for a further 10 seconds.



Hold the foam tip against the skin tag surrounded by the protective plaster for 40 seconds.



Remove the protective plaster directly after treatment.



Remove the foam tip from the applicator cap opening by pulling it out by its holder. Dispose accordingly.

In case of multiple skin tags located close to each other, treat no more than two skin tags at a time. Foam tips provided are single use disposable items and not for reuse. For each skin tag to be treated use a new foam tip.

Do not touch the surface of the used foam tip. Hold the foam tip away from other skin surfaces. Contact with the used foam tip surface could result in freezing of the skin surface.

4. AFTER THE TREATMENT

- During or after treatment a slight pain or pricking sensation can occur minor swelling of the treated skin tag can also be a normal side effect of cryotherapy
- The treatment may cause blistering. If they do treat as normal blister
- Remove the protective plaster, do not touch or scratch the treatment area
- Keep the treatment area clean and avoid infection
- Swimming and taking a shower or bath is allowed. Do not scrub the treated area
- The skin tag will fall off after new healthy skin has formed under the treated area

5. SIDE EFFECTS

Not following the instructions for use may result in frostbite, skin damage or scarring. The freezing process may cause light or dark patches on the skin. This is called hypo- or hyper pigmentation and can be one of the side effects of cryotherapy. This can be a higher risk when used on heavily pigmented skin and/or by not following the instructions for use carefully. When used as instructed, these patches will gradually disappear. Using the product in a way other than described in this instructions for use, can lead to thermal reactions, hypo- or hyper pigmentation, damage of the skin and scar formation. Consult a doctor in case of infection and/or complications.

6. CONTRAINDICATIONS

Do not use HeltiQ Skintags:

- When there is or has been a confirmed diagnosis of cancer and/or treatment for cancer
- Skin tags larger than 4 mm. Large skintags are not treatable with this method
- When the colour of the skin tag differs from the colour of the surrounding skin
- Skin tags on buttocks and ano genital warts
- Heavily pigmented skin / dark pigmented skin
- When the skin tag is hairy or contains hair
- When the skin tag is hard
- When the skin tag bleeds
- Irritated, infected or sensitive skin
- When there is sensitivity or allergy to extreme cold (cold type urticaria)
- Diabetics and persons with circulatory problems including Raynaud's disease
- Patients treated with immunosuppressive medicines/confirmed diagnosis for autoimmune or collagen diseases
- Patients on anticoagulation medication and haemophilia patients
- On children under 4 years of age
- HIV infected patients
- In combination with other treatments for skin tags
- On children under 4 years of age
- Warts on face, chest, neck, breasts or buttocks

7. WARNINGS AND PRECAUTIONS

- Extremely flammable aerosol Pressurized container: May burst if heated
- Keep out of reach of children
- Do not breathe spray
- Keep away from heat, hot surfaces, sparks, open flames and other ignition sources
- No smoking
- Do not pierce or burn, even after use
- Do not spray on an open flame or other ignition source
- Protect from sunlight
- Do not expose the aerosol dispenser to temperatures exceeding 50 °C / 122 °F
- Store in a cool and dry area at maximum room temperature
- Dispose of contents/container in accordance with local regulations

If you intend to use HeltiQ Skintags while you are pregnant or breastfeeding, consult a physician in advance. In all cases of doubt consult your physician.

COMPLAINTS AND CUSTOMER SERVICE SUPPORT

Contact the manufacturer or supplier with questions or to report complaints.
www.skintag.eu



Since 1880

Manufactured by:

Koninklijke Utermöhlen N.V.
De Overweg 1, 8471 ZA Wolvega,
The Netherlands +31 (0)561 69 33 66
www.skintag.eu



0344